VOCO

x-tra base

(GB)

in accordance with DIN EN ISO 4049

Instructions for use

(CE) 0482

Product description:
x-tra base è un radiopaque, light-curing flowable composite for base fillings. In contrast to conventional flowables, it is characterized by an increased depth of cure while at the same time showing reduced shrinkage stress and can be cured in layers of 4 mm thickness.

x-tra base has to be finished occlusal with an at least 2 mm thick surface layer of a methacrylate-based universal/posterior composite. **x-tra base** is used with a dentine-enamel bond. **x-tra base** contains 75% w/w inorganic fillers in a methacrylate matrix.

Shades:
Universal, A2

Indications:

- Base in class I and II cavities
- Cavity lining under direct restorative materials in class I and II cavities

Type of application:

Shade selection:

To select a shade of the final composite chosen for the occlusal surface, compare its shade guide with the cleaned tooth before anaesthesia. Moisten the shade sample and tooth and select the shade, preferably in daylight.

Creating a dry working field:

Ensure that the work area is sufficiently dry. Use of a rubberdam is recommended.

Cavity preparation:

In principle, cavity preparation should be carried out according to the rules of adhesive filling therapy and should be minimally invasive to conserve a healthy tooth substance. Clean and dry the cavity afterwards.

Matrix placement:

Take appropriate measures for areas where the applied **x-tra base** can flow out. Matrices that are wedged in the approximal area are advantageous. Minimal separation facilitates the shaping of the approximal contact and insertion of the matrix.

Pulp protection:

A suitable pulp-protective liner should be applied in close proximity to the pulp and can be covered with a stable cement.

Bonding material:

x-tra base is used in the adhesive technique with a dentine-enamel bond. All light-curing bond materials can be used. Follow the respective instructions for use with regard to preparation (etching technique) and application.

Application:

Syringe: Lock the enclosed intraoral tip on the syringe with a clockwise turn and directly apply **x-tra base**. Do not pull the plunger back on the syringe during or after use to guarantee the function of the non-dripping syringe.

Caps: Remove protective cap. Insert Caps into the opening of the dispenser (following the instructions for use).

Bend the metal cannula into the desired direction. Begin applying the material directly into the prepared cavity at the deepest point and fill the cavity from bottom to top. Keep the tip dispensing at all of the time. To prevent the formation of air bubbles ensure a slow, even flow out. If necessary, air bubbles can be removed before polymerisation. **The distance to the occlusal margin of the cavity must be at least 2 mm.** **x-tra base** can be polymerised in increments up to 4 mm. Alternatively a thin layer of x-tra base can be applied as cavity liner.

Light-curing:

Conventional polymerisation devices are suitable for light-curing the material. Make sure that all areas of the filling should be cured. An additional curing of the accessible areas from the oral or vestibular side is recommended. If metal matrices are used, these should be removed prior to additional curing. Hold the light emission tip of the device as close as possible to the surface of the filling. Otherwise, the curing depth may be reduced. Incomplete curing may lead to discolouration of the material, loss of stability of the composite filling and discomfort.

Depending on the light-output and the chosen color the curing times are:

Shade	universal	A2
Light-output		
500 - 800 mW/cm²	10 s	40 s
> 800 mW/cm²	10 s	20 s

Final occlusal layer:

x-tra base has to be finished with an at least 2 mm thick layer of a methacrylate-based universal/posterior composite. For proceeding and finishing follow the manufacturer's instructions for use.

Information/precautionary measures:

- x-tra base** should not be used in case of known allergies to any of its ingredients (methacrylates, BHT).
- Allow the material to reach room temperature before use.
- Phenolic substances, especially preparations containing eugenol or thymol, can interfere with the curing of filling composites. Avoid the use of zinc oxide eugenol cements or other eugenol-containing materials in combination with filling composites.
- The enclosed intraoral tips are intended for a single use and must be disposed of after only one use. Return the cap to the syringe after use.
- Use each **x-tra base** Caps for one patient only!

Storage:

Store at temperatures of 4°C to 23°C. To avoid exposure to light and potentially resulting polymerisation, syringes should be closed immediately after dispensing. Do not use **x-tra base** after the expiry date.

Our preparations have been developed for use in dentistry. As far as the application of the products delivered by us is concerned, our verbal and/or written information has been given to the best of our knowledge and without obligation. Our information and/or advice do not relieve you from examining the materials delivered by us as to their suitability for the intended purposes of application. As the application of our preparations is beyond our control, the user is fully responsible for the application. Of course, we guarantee the quality of our preparations in accordance with the existing standards and corresponding to the conditions as stipulated in our general terms of sale and delivery.

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